

# Using healthPAL™ with the AND767PBT Blood Pressure Monitor



From the MedApps HealthPAL QuickStart Guide

## Collecting Blood Pressure Readings

1. Ensure that blood pressure cuff is connected to the blood pressure monitor.
2. Place blood pressure cuff on upper arm.
3. Power on the blood pressure monitor by pressing the START button located on the face of the monitor.
4. The monitor will display the previous reading; followed by a blinking "0" (zero), initiating the inflation of the blood pressure cuff.
5. Monitor will display new reading after which HealthPAL will visually display and audibly announce the current reading. (**"Your blood pressure reading is XXX."**)

The measurement will be stored in HealthPAL's memory and transmitted to server automatically when HealthPAL is inactive for 3 minutes.

*Always follow manufacturer instructions for operation and safety for any accessory health monitors used with HealthPAL.*

**"Your Blood Pressure is: One-Hundred Twenty Eight Over Eighty Five."**

